

THE PURPLE™ STIRRUP REPAIR GUIDE

03-0106-A UPHOLSTERED BOOT PAD


Scope: This document provides instructions for the replacement of the Upholstered Boot Pad on the Kyra Comfort 3000, 3100, 5000, 5100 and similar models.

BEFORE USING THIS PROCEDURE ENSURE THAT IT IS THE LATEST REVISION:

03-0106, Rev A

Kyra Repair Guide_Boot Pad
ECO 1053, Effective Date: 28 June, 2019

	Kyra Medical Inc. 100 Otis St. Unit 1 Northborough, MA 01532 United States of America	
	EC REP Emergo Europe, Prinsessegracht 20 2514 AP The Hague, Netherlands	

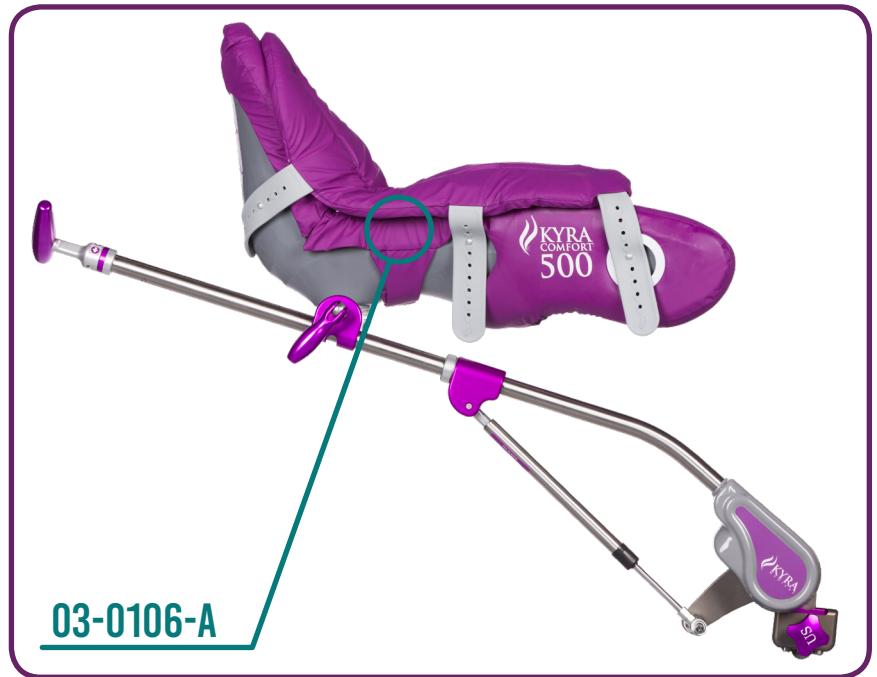

REQUIRED COMPONENTS, OTS PARTS AND MATERIALS


ITEM	PART NUMBER	DESCRIPTION	QTY.
1	KYRA9XXX	Refer to Repair Kit Order Form, doc. #MKT-1012-A for specific Upholstered Pad type	1
2	OTS-1080	Loctite 222mS, Threadlocker, One-time use, .02 oz	1

Property of Kyra Medical, Inc. ©2019



1. REMOVAL INSTRUCTIONS, ALL TYPES:

1.1.a. Unfasten the Strap on the underside of the Boot Shell. (fig. 1.1.a)

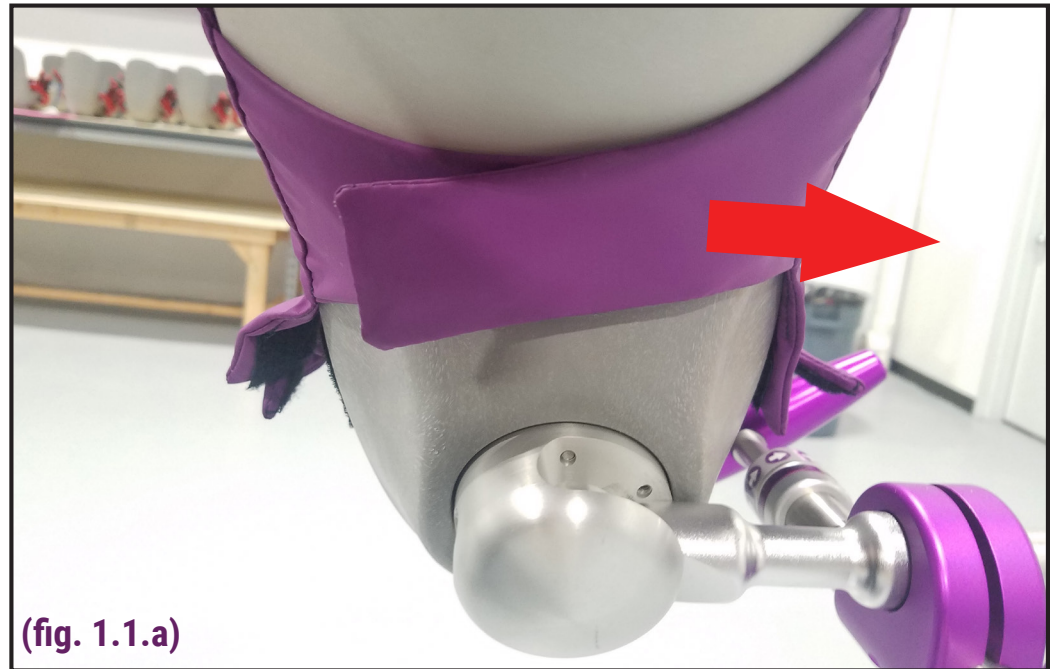

1.1.b. Pull both angled tabs from Ankle location on both the Lateral and Medial sides to release from the Boot Shell. (fig. 1.1.b)



1. REMOVAL INSTRUCTIONS, ALL TYPES: (CONT'D)

1.1.c. Begin removing the Boot Pad at the foot section . (fig. 1.1.c)



1.1.d. Reach under the Wing section of the Boot Shell and release the Velcro™. (fig. 1.1.d)



1. REMOVAL INSTRUCTIONS, ALL TYPES: (CONT'D)

1.1.e. Continue removing the Boot Pad by removing the Wing sections. (fig. 1.1.e)




1.1.f. With Secure-Lok™ Strap variants, take care not to damage the grommet when passing over the Strap Post. (fig. 1.1.f)



1.1.g. KYRA3000 and KYRA3100 Clam-shell pad types are now removed, proceed to the next section for KYRA5000 and KYRA5100 Top Pad removal instructions.

1. REMOVAL INSTRUCTIONS, TOP PAD:

- 1.2.a. Remove two 10-32 Button Head Screws, one at each Anchor Strap location on the Medial side. Retain all Screws and Washers for Installation steps. (fig. 1.2.a)
Remove Top Pad.

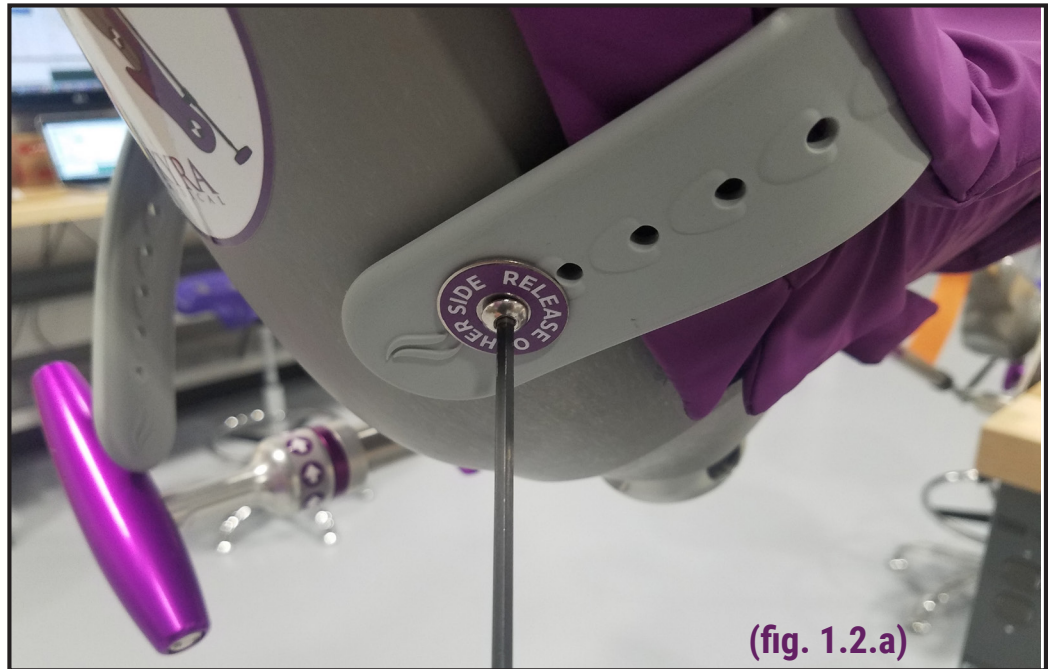

Continue to Installation Instructions...

2. INSTALLATION INSTRUCTIONS, ALL TYPES:

2.1.a Remove new Boot Pad from packaging. Tilt the boot so the wings are facing up. Slide the long wing part of the pad onto the long wing and pull downward until seated. Take care to prevent the plastic grommet from snagging on the Strap Post with Secure-Lok™ variants. (fig. 2.1.a)



2.1.b Pull the short wing side over the short wing and pull downward until seated. (fig. 2.1.b)




2.1.c Pull downward between the wings and press against the Velcro™ piece on the underside of the boot. Pad should lay flat in this area. (fig. 2.1.c)



2. INSTALLATION INSTRUCTIONS, ALL TYPES: (CONT'D)


2.1.d Final under-knee pad area should be snug to the boot with minimal wrinkles. (fig. 2.1.d)

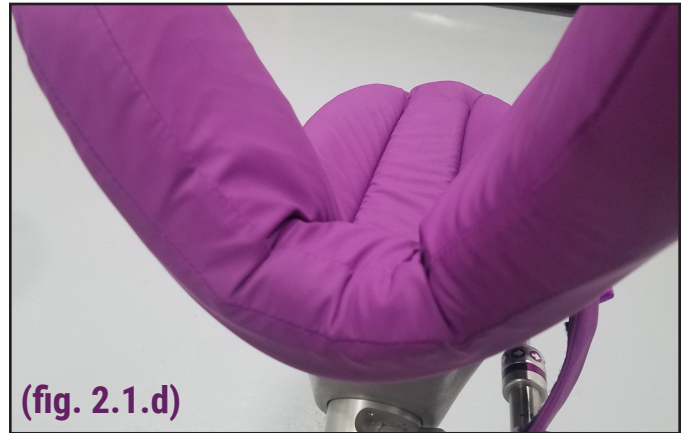

2.1.e Without engaging the two areas of Velcro in the boot cavity, pull the Toe area over the toe of the boot. (fig. 2.1.e)




2.1.f Push down on central area of the foot section and then the wing section toward the heel area. This will engage the Velcro underneath and minimize wrinkling. (fig. 2.1.f)

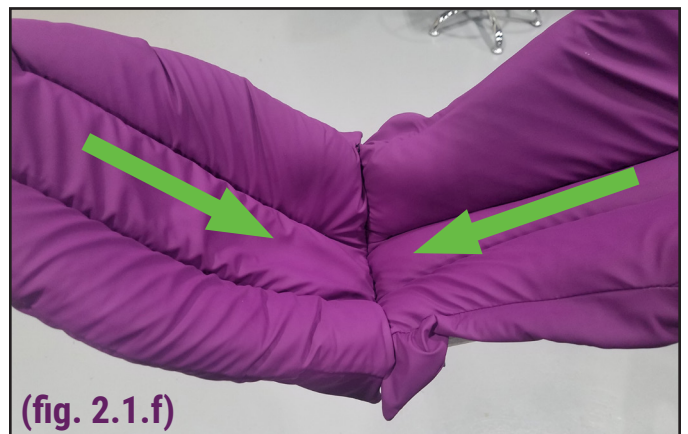

2. INSTALLATION INSTRUCTIONS, ALL TYPES: (CONT'D)

2.1.g Pull Two Side Straps together under the boot. Ensure the Velcro mate is straight and even.
(fig. 2.1.g)

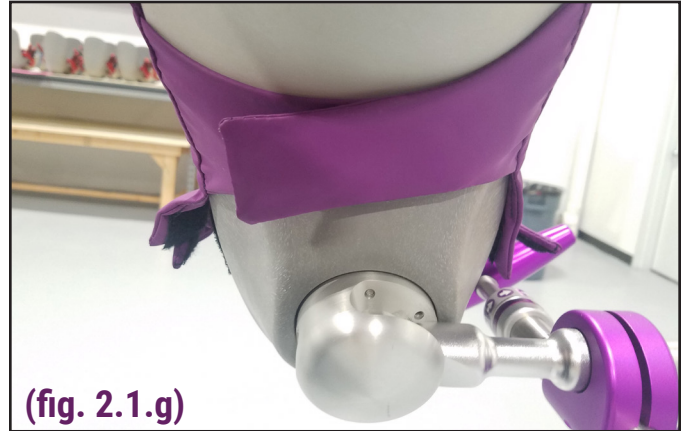

2.1.h In the Ankle location, attach the sides of the pad to the curved Velcro pieces on the sides of the boot. Corners should align evenly. Repeat on opposite side.
(fig. 2.1.h)



Clamshell top flap design and is now complete.

Proceed to Step 3 if using a Boot Pad style with a separate Top Pad.

3. INSTALLATION INSTRUCTIONS, TOP PAD:


3.1.a Place the Top Pad onto the boot. Orient the curved side at the Toe and the angle pointing at the long wing. Two anchor straps face the short fin and 3 silicone or Velcro™ straps face the long wing. (fig. 3.1.a)



3.1.b Secure Anchor Straps to Short Wing side: Place washer with “Release Other Side” warning label facing outward. Insert the 10-32 x 3/8” button head hex screw through the washer and into the strap. Hold in position for the Loctite application step below. (fig. 3.1.b)




3.1.c Apply Loctite 222MS (purple) to the screw threads. (fig. 3.1.c)

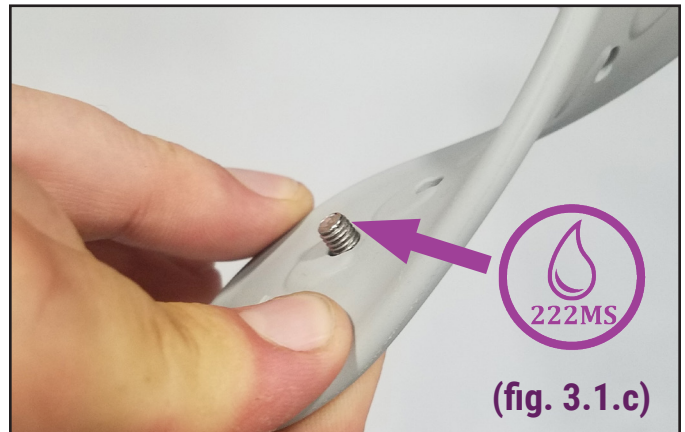

3. INSTALLATION INSTRUCTIONS, TOP PAD: (CONT'D)

3.2.a Thread screw, washer and anchor strap into corresponding insert on Short Wing side of boot. Do not over tighten, Silicone strap should only slightly deflect. Repeat Step 3.1.b-3.2.a with remaining strap. (fig. 3.2.a) Straps should not be twisted when finished.

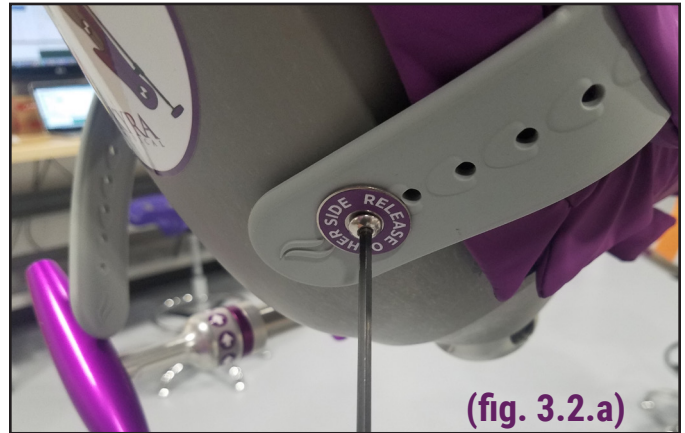

END