


Integra MLX Lamp Replacement Guide (CL300BF)



REMOVE SCREWS AT THE ARROWS BELOW AND SLIDE OFF LID



REMOVE BULB MODULE RETENTION PLATE BY LOOSENING THE SCREW SHOWN BY THE ARROW BELOW

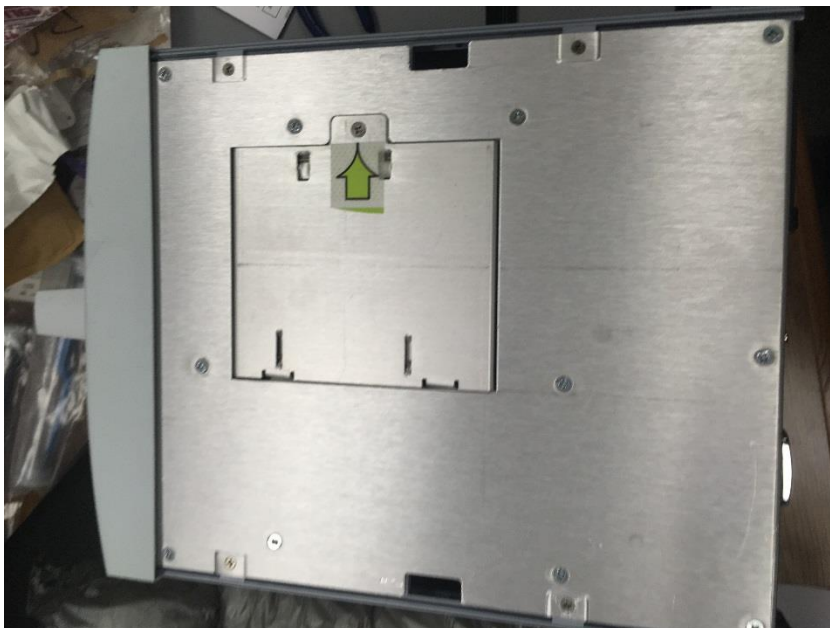

Integra MLX Lamp Replacement Guide (CL300BF)



ONCE SCREW IS LOOSENED, THE PLATE IS EASILY REMOVED



MLX MODULE IS NOW READY TO REMOVE



Integra MLX Lamp Replacement Guide (CL300BF)



SIMPLY GRAB THE MODULE AND PULL OUT



Integra MLX Lamp Replacement Guide (CL300BF)



FLIP THE MODULE AND YOU'LL SEE THE TWO GOLD PLUG PINS. THESE PINS KEEP THE PLASTIC OUTER HOUSING AND THE METAL INNER HOUSING TOGETHER



UNSCREW THE GOLD PLUG PINS




Integra MLX Lamp Replacement Guide (CL300BF)



SLIDE THE METAL BULB HOUSING FROM THE OUTER PLASTIC HOUSING



THE BULB IS LOCKED IN THE METAL HOUSING BY TWO HEX SCREWS

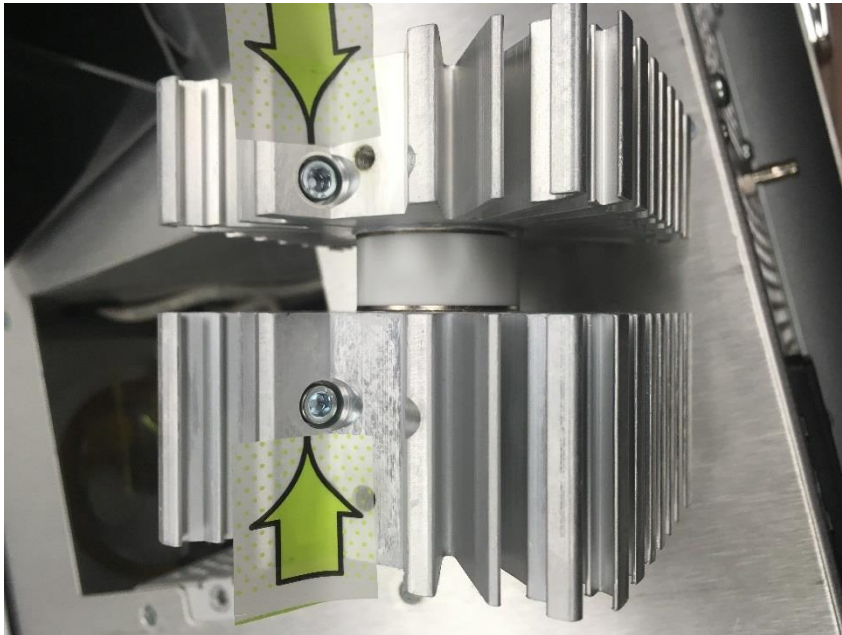

Integra MLX Lamp Replacement Guide (CL300BF)



LOOSEN THESE SCREWS USING A 9/64" HEX WRENCH




ONCE LOOSE, THE HOUSING CAN BE PULLED APART



Integra MLX Lamp Replacement Guide (CL300BF)



BULB WILL NOW BE EXPOSED AND YOU CAN REMOVE AS SHOWN

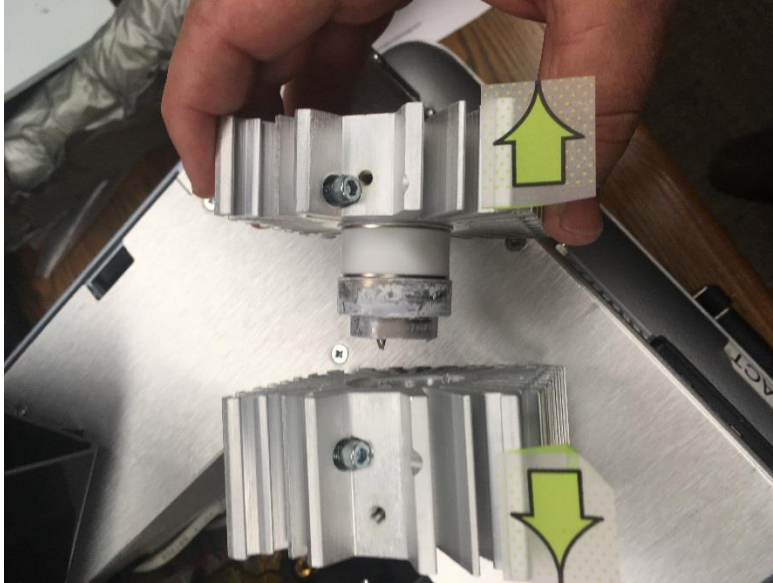


Integra MLX Lamp Replacement Guide (CL300BF)



Integra MLX Lamp Replacement Guide (CL300BF)





TAKE NEW CL300BF AND GREASE BOTH ENDS

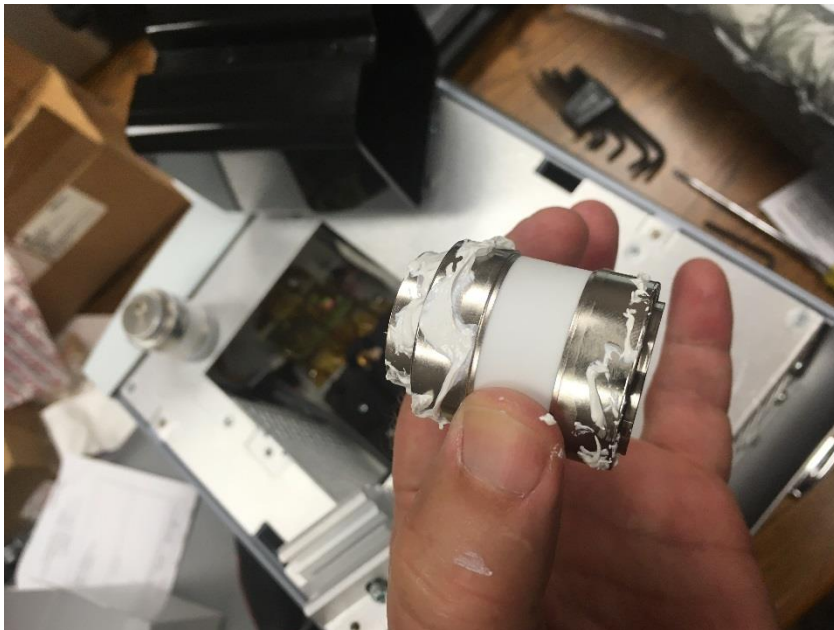

Integra MLX Lamp Replacement Guide (CL300BF)




PLACE BULB BACK IN ITS METAL HOUSING AS SHOWN

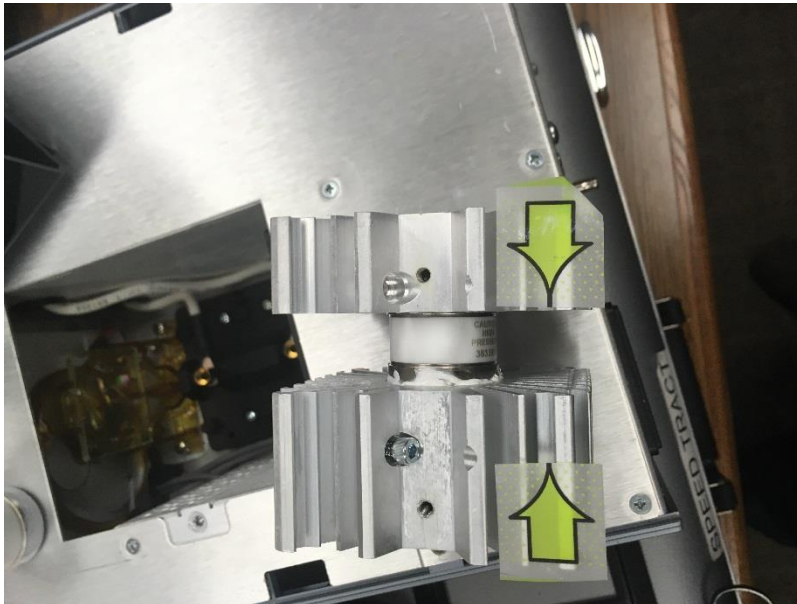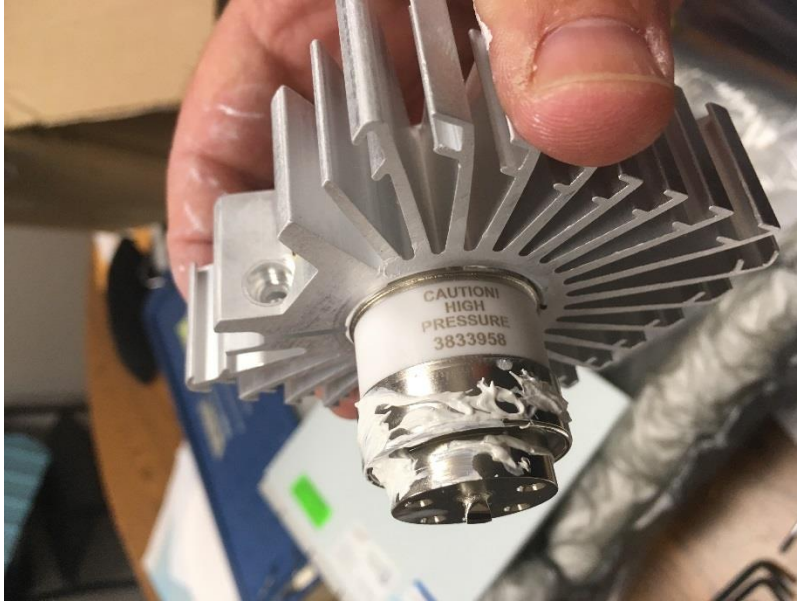

Integra MLX Lamp Replacement Guide (CL300BF)





TIGHTEN METAL BULB HOUSING SCREWS AS SHOWN

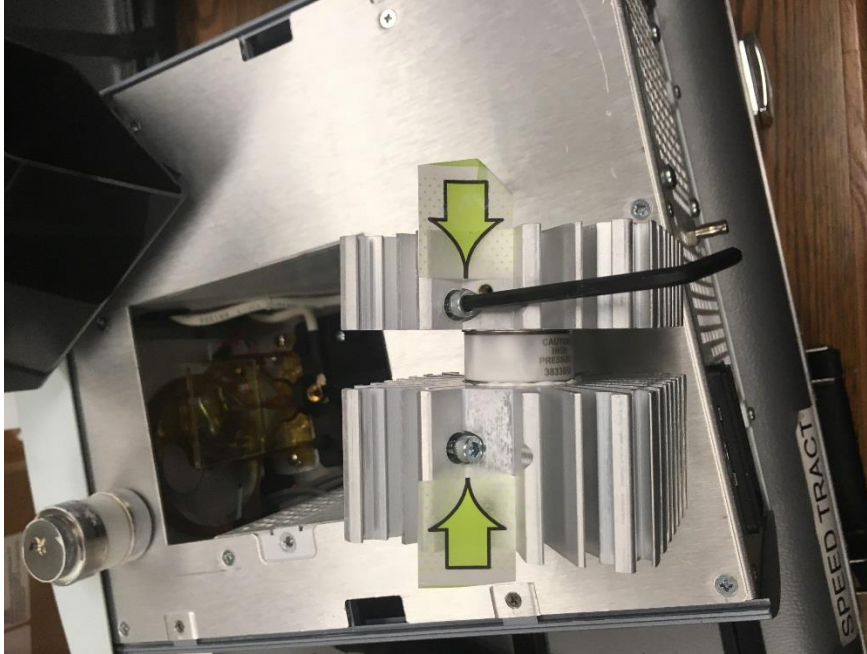

Integra MLX Lamp Replacement Guide (CL300BF)




REPLACE METAL HOUSING INSIDE THE PLASTIC OUTER HOUSING AND LINE UP THE HOLES FOR THE GOLD PLUG PINS AS SHOWN BELOW

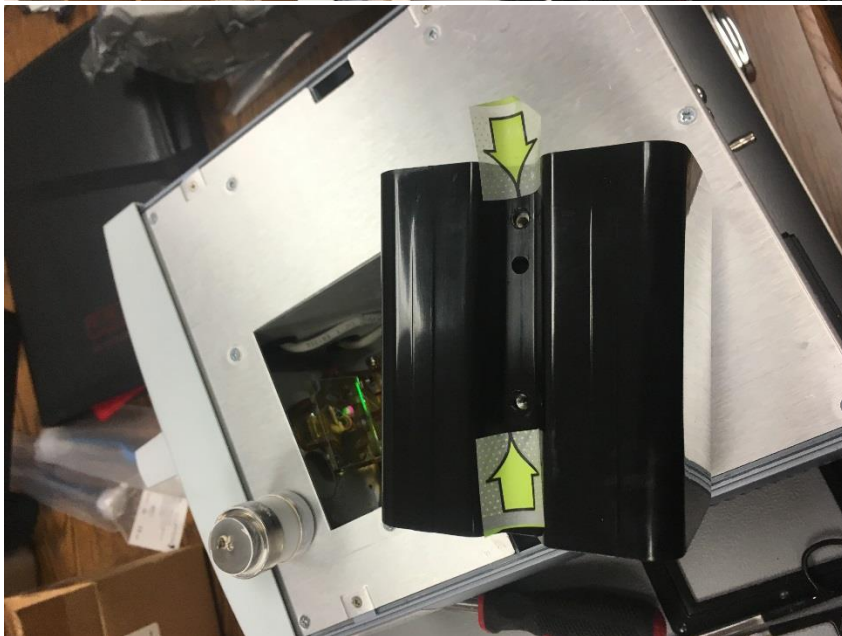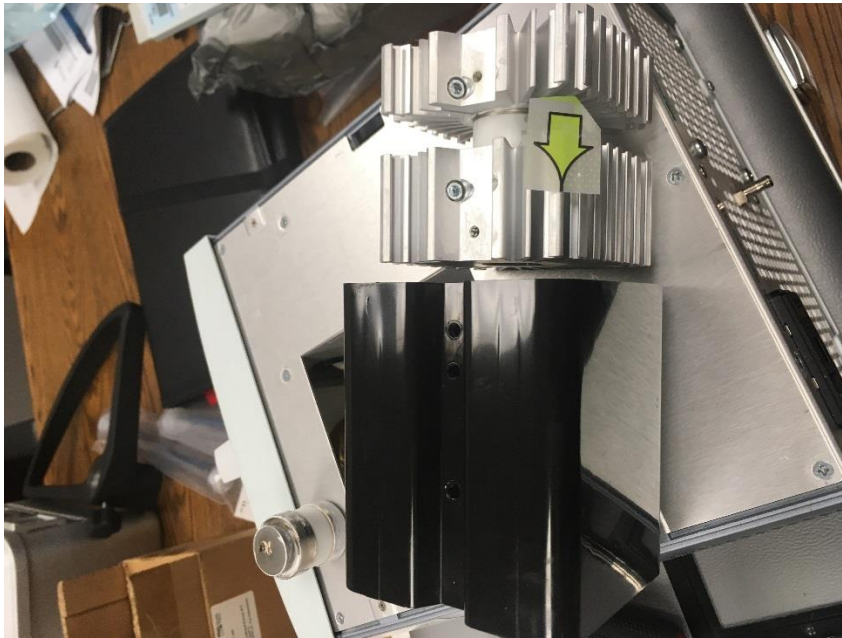

Integra MLX Lamp Replacement Guide (CL300BF)



REPLACE THE GOLD PLUG PINS...THE BULB MODULE IS NOW FULLY ASSEMBLED

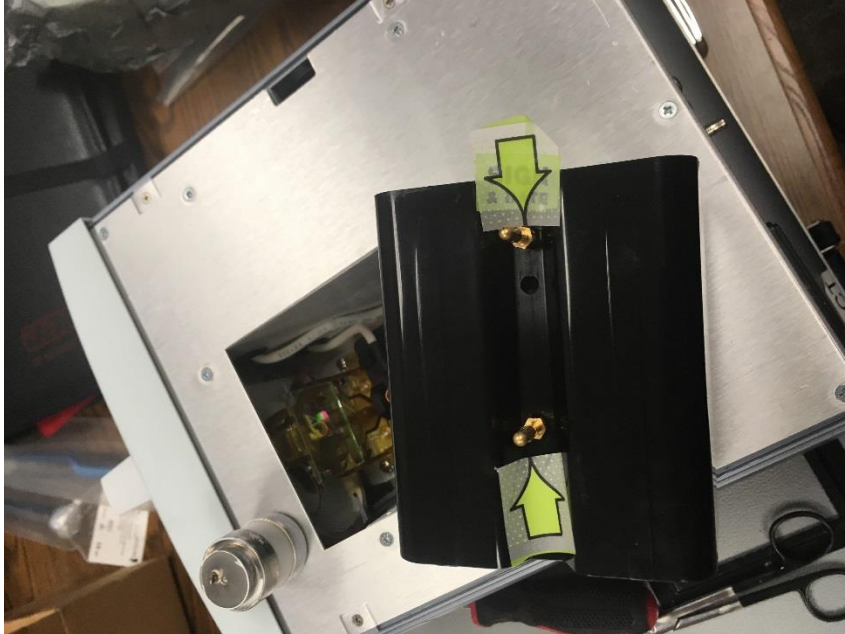

Integra MLX Lamp Replacement Guide (CL300BF)



REPLACE MODULE IN THE MLX AS SHOWN



Integra MLX Lamp Replacement Guide (CL300BF)



REPLACE MODULE COVER PLATE AND TIGHTEN SCREW



Integra MLX Lamp Replacement Guide (CL300BF)



SLIDE MLX COVER IN PLACE AND TIGHTEN COVER RETENTION SCREWS AS SHOWN



BULB REPLACEMENT IS NOW COMPLETE.

Integra MLX Lamp Replacement Guide (CL300BF)



To Reset Lamp Age Meter:

1. Using the membrane switch, turn the display to the lamp age/system age screen.
2. On the rear of the unit insert a pen/pencil or other suitable pointed object into the slot marked RESET (Figure 7).
3. Push until a click is felt.
4. Verify that the lamp age has been reset to 0.

NOTE : The System Age reading CANNOT be reset.